GB

# HILARY<sup>®</sup>

## FRÜH

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#### HOW TO CARRY OUT THE TEST

##### MIDSTREAM METHOD

1. Take the test from the package in order to use it.  
Note: If the test was kept in the refrigerator, it has to be brought to room temperature first before using it.
2. Remove the cap of the test to see the aspiration rod.
3. Hold the test at the grip mark; the aspiration rod must point downward.
4. Hold the aspiration rod directly in the urine stream for about 5 seconds. The aspiration rod must be completely moistened with urine.
5. In order to achieve a correct result, place the test (with the result window pointing upward) on a level surface. Pink lines will become visible in the result window.
6. Read the result after 5 minutes.

##### DIP METHOD

1. Collect urine in a clean and dry jar.
2. Take the test from the package in order to use it.  
Note: If the test was kept in the refrigerator, it has to be brought to room temperature first before using it.
3. Remove the cap of the test to see the aspiration rod.
4. Dip the aspiration rod into the jar containing the urine for about 5 seconds. The aspiration rod must be completely moistened with urine.
5. In order to achieve a correct result, place the test (with the result window pointing upward) on a level surface. Pink lines will become visible in the result window.
6. Read the result after 5 minutes.

#### RESULT ASSESSMENT

##### 1. Negative

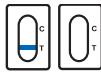
The pink control line (C) is visible, but there is no pink test line (T). This means that you are not pregnant.

Note: If your menstruation does not occur within a week's time, repeat the test. If you get a negative result again, please contact your doctor.

##### 2. Positive

Both the pink control line (C) and the pink test line (T) become visible. This means that you are pregnant. One line may be thicker/thinner or more intensive than the other, but always pink. You should see two clear parallel lines. In case of an early proof of pregnancy in particular, when the hCG concentration is below 25 mIU/mL, the test line (T) may be very thin.

Note: Please contact your doctor in order to inform him/her of your pregnancy and agree upon the next step. Early prenatal examinations are important in order to guarantee the good health of yourself and your baby.

##### 3. Invalid

The result is invalid, if no pink control line (C) becomes visible within 5 minutes. If you get an invalid result, you did either not follow the instructions correctly or the test does not function properly. Please repeat the procedure using a new test. If the problem persists, please contact your local distribution partner and/or your doctor.

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DOLORGIET GmbH u. Co. KG
Humasis Co., Ltd. · Rm.504, SHINWON VISION TOWER
MT Promedt Consulting GmbH